



The Portable Nebulisation System for Horses

### Simple & integrated

Easy set up and operation no hoses, wires or awkward valves

### Rapid

10mls saline aerosolised in less than 12 minutes

### Silent

Noise Free

### Adaptable

For use with MDIs/inhalers



From Propress Equine the manufacturers of HAYGAIN hay steamers



For more information contact the UK sole distributor Propress Equine:  
Propress Equine Ltd  
1 Leverton Buildings, Hungerford,  
Berkshire, RG17 0TA  
T: 0333 200 5233  
E: [info@propressequine.co.uk](mailto:info@propressequine.co.uk)  
W: [www.propressequine.co.uk](http://www.propressequine.co.uk)



fast, portable,  
equine  
nebulisation  
device





Flexineb™ is a fast, portable equine nebuliser device that is simple to use and noise free. It operates without the noise of loud compressors or the inconvenience of cumbersome hoses and pipes.

Flexineb utilises Nortev proprietary technology and is supremely portable allowing in excess of 3 hours of operation before the Lithium Ion battery needs recharging.

### Drugs:

The Nortev Flexineb system is designed to aerosolise a wide range of drugs including corticosteroids, bronchodilators, saline and antibiotics.

Medications must be prescribed by veterinarians; end users should follow the advice of veterinarians at all times.

### Cleaning & Disinfecting:

Cleaning is easy - simply wash with warm soapy water. Disinfecting is simple too, by using cold sterilisation methods.

### Volume of Medication:

A standard 10mls medication cup is accommodated. A 20mls extension is provided to facilitate volumes of up to 30mls. No medication remains in the cup after treatment.

### Comfort & Tolerance:

Flexineb fits comfortably onto the horse's nose as a result of the flexible material. An adjustable lever allows full control of the exit valve, enabling the caregiver to bypass the valve for easy breathing or to facilitate inhalation deep into the lungs.

### Visibility:

The transparent aerosol holding chamber allows the user to see the aerosol mist being generated by the Flexineb technology, in conjunction with the horse's breathing emptying the chamber and receiving the fine particle aerosol.

### Accessories:

System accessories include an AC/DC power supply to charge the internal battery, along with a user manual and a convenient storage bag.



1

Portable, easy to assemble mask



2

Battery operated controller



3

Nebuliser

